

NOTICE D'UTILISATION

INSTRUCTION FOR USE

MAGiC
SONIC



FR **Magic Sonic** réduit les bruits stridents en amortissant la pression acoustique dans la gamme de fréquences autour de 6000 Hz. L'environnement sonore et la perception des sons de la parole restent inchangés pour le confort des chirurgiens-dentistes, hygiénistes, assistant(e)s et patients.

Conseils d'utilisation : Placer **Magic Sonic** à l'entrée du conduit auditif en le présentant par l'extrémité la plus large, sans l'enfoncer (R : oreille droite - L : oreille gauche) (cf. photo #1). Ajuster **Magic Sonic** en le tournant pour que son positionnement soit confortable et qu'il épouse la forme de la conque afin d'obtenir des performances optimales. Nettoyer **Magic Sonic** régulièrement et le conserver dans son emballage d'origine.

Nettoyage : Séparer les deux parties (cf. photo #2) pour un nettoyage optimal. Immerger et traiter dans un nettoyeur à ultrasons selon les préconisations du fabricant de la solution nettoyante (concentration, température) pendant un minimum de 5 minutes. Rincer à l'eau courante puis sécher à l'air comprimé. Réassembler les deux parties correspondantes avant utilisation (cf. photo #3). **Magic Sonic** supporte la stérilisation à la vapeur en autoclave.

UK **Magic Sonic** reduces shrill noises by dampening sound frequencies of around 6000 Hz, while usual environmental sounds and speech remain unchanged. Ideal for dentists, hygienists, dental assistants and patients.

Instructions for use : Carefully place **Magic Sonic** into the ear canal, presenting by the widest end (R: right ear - L: left ear) (pic #1). Adjust **Magic Sonic** by rotating it so that its positioning is comfortable and that it fits the shape of the conch to obtain optimal performances. Clean **Magic Sonic** regularly and keep it in its original packaging.


Cleaning : Separate the two parts (pic #2) and soak in an ultrasonic bath according to the cleaning solution manufacturer's instructions (concentration, temperature) for a minimum of 5 minutes. Rinse with water then dry with compressed air. Reassemble the two parts before use (pic #3). **Magic Sonic** can be sterilized in autoclave.



#1



#2

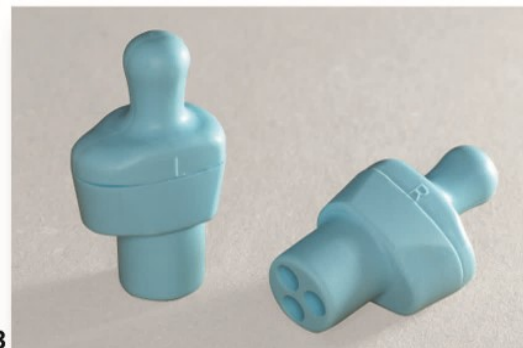

#3

